



[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

This medical device is intended for patient positioning. This is not a restraint system.

Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

## INTEGRAL BIRDY BELT

FOR ALL ARMCHAIRS



Maximum useful scope: + 220 cm  
Buckle: 50 mm

It can be set up with the patient seated and avoids any pelvic constriction.

Its shoulder heights are separately adjustable to help position and compensate for a bad position.

Available in 4 sizes S, M, L, XL

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
Waist (in cm)



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



CBRIN2HP-PF



v 07.02/25

### Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



# INTEGRAL BIRDY BELT

## HOSPITAL RANGE

Combination of polyurethan foamand  
on both faces

Jersey/PU coating

100% polyester bias

100% polyester strap

Nylon buckle

Thermo compressed foam

1. Slip the head through the shoulders harness.
2. Place the Birdy belt in the patient's back at the lumbar and adjust the small loops on each hip.
3. Pass the end of the belt under the slightly raised thigh.
4. Do the same for the second thigh.
5. Pass each buckle in the corresponding loop.
6. Set up the abdominal belt with the velcro strap.
7. Place the double buckle in the back strap.
8. Close the buckle at the back of the chair.
9. At the back, set up the two rear buckles of the harness.

