



www.nausicaa-medical.com

This medical device is intended for patient positioning. This is not a restraint system.

Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

SIMPLE BIRDY BELT

FOR ALL ARMCHAIRS - SPECIAL SHELL



TO USE ONLY FOR THE INDICATED USE:
PREVENT SLIDING
FROM THE BASIN TO THE
FRONT

Maximum useful scope: + 220 cm
Buckle: 50 mm

It can be set up with the patient seated and avoids any pelvic constriction.

Available in 4 sizes S, M, L, XL

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
Waist (in cm)



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



CBR2HP-PF

v 07.02/25

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



SIMPLE BIRDY BELT

HOSPITAL RANGE
Coated polyester jersey
doubled polyurethane
polyurethane foam
Bias 100% polyester
100% polyester straps
Nylon buckles

to use only for the indicated use:
PREVENT BASIN SLIDING FORWARD

1. Slide the Birdy belt into the back of the patient at the lumbar level and adjust passersby on each hip.
2. Pass the end of the belt under the leg slightly raised.
3. Replicate the operation on the second thigh.
4. Pass each loop through the loop related.
5. Close the buckle on the back of the chair.

