



www.nausicaa-medical.com

This medical device is intended for patient holding and positioning. It is not a restraint system.
Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

Y SLIM BUST BELT

FOR ALL CHAIRS



TO MAINTAIN THE CHEST IN
UPRIGHT POSITION



Maximum useful area: + 230 cm
Height of the belt: + 11 cm
Width strap: + 48 mm
Buckle: 50 mm

The shoulder heights are separately adjustable to help position and compensate for a wrong position.

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
- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm
- Waist (in cm)



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



CSY2HP-PF

v 07.02/25

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



Y SLIM BUST BELT

HOSPITAL RANGE

Combination of polyurethan foam and
on both faces jersey/PU coating.

100% polyester bias

100% polyester strap

nylon buckle

thermo compressed foam

- Place the shoulder belt being careful that the shoulders pads are positioned on the patient's shoulders, buckles behind the backrest of the chair.
- Place the belt around patient's waist and close the belt through the velcro strap.
- Grasp the patient and the backrest of the chair with the abdominal strap.
- Close the buckle, adjust the length of the strap while being careful not to compress the abdomen of the patient.
- Close the buckles of the shoulder belt and adjust their length (see front page).