



www.nausicaa-medical.com

This medical device is intended for patient positioning. This is not a restraint system.

Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fasteners.

BUST JACKET

FOR ALL CHAIRS



TO MAINTAIN THE CHEST IN
UPRIGHT POSITION
FOR ALL CHAIRS

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
Maximum useful scope: + 220 cm
Buckle: 40 & 25 mm

Made in a breathable fabric, it is easy to put in place thanks to its unbreakable shoulders.

- Declined in 3 sizes S, M, L corresponding to the sizes of clothing.
- Maximum useful area: 220 cm



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



CGI3-PF

v 07.02/25

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



BUST JACKET

HEMOCARE RANGE
Bi-laminate warp and weft Polyester
fabric with Polyurethane membrane
100% cotton bias
100% polyester strap
Nylon buckle

- Open the buckles situated on the shoulders.
- Place the complete jacket.
- Close the buckles.
- Cross the jacket on the front.
- Pass the belt strap through the provided slit.
- Grasp the backrest of the chair while being careful not to compress the abdomen of the patient.
- Pass the end of the additional delivered strap in the back loop.
- Close the 40mm buckle.
- Adjust the length of the strap through the 25mm nylon buckle to avoid that the person falls forward.