

www.nausicaa-medical.com

This medical device is intended for patient positioning. This is not a restraint system. Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fasteners.

YZA JACKET

FOR ALL CHAIRS

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


A vest, plus a single size pelvic module, allow you to go from a bust fit to an integral fit according to changing needs without size constraints.

- Maximum useful area: 200 cm



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



YZA2GBHP-PF

v 07.02/25

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



YZA JACKET

HOSPITAL RANGE

Polyester-coated polyester jersey
lined with polyurethane foam

Bias 91% polyamide - 9% elastane

100% polyester webbing

PVC Shoulder Pad

Nylon buckle

DO NOT USE THIS STRAP IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

The YZA jacket comes in 2 modular variants:

1-YZA bust jacket (YZA-GB)

- a - The pelvic part must be returned inside the vest
- b - Put on the bust vest, positioning the shoulder pads well on the shoulders of the person, the buckles behind the back of the chair.
- c - Position the belt at the waist of the person.
- d - Gird the patient and the back of the chair with the lap strap.
- e - Close the loop and adjust the length, taking care not to compress the abdomen.
- f - Pass a shoulder strap under the abdominal support strap, connect the loops, and adjust its length.

2 -The YZA integral jacket (YZA-GI) consists of the YZA Bust Jacket to which is added a pelvic support module.

- a - Position the pelvic support module on the chair seat, and secure the 25 mm strap to the back of the chair with minimum play.
- b - Sit the patient on the chair and on the strap.
- c - Put in place the bust vest, positioning the shoulder pads well on the shoulders of the person, the loops behind the back of the chair.
- d - Pass the straps behind the back of the chair.
- e - Close the nylon loop of the lap strap, then adjust the length of the strap, being careful not to compress the patient's abdomen.
- f - Pass the strap of the pelvic support in the passer of the pelvic part. Close the 50 mm loop of the pelvic support by adjusting its length.