



www.nausicaa-medical.com

This medical device is intended for patient holding and positioning. It is not a restraint system. Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

## SLIM FIT

FOR ALL CHAIRS



TO PREVENT FORWARD  
SLIDING AND MAINTAIN THE  
CHEST IN UPRIGHT POSITION  
FOR ALL CHAIRS EXCEPT  
SHELLS



Maximum useful area: + 230 cm  
Belt height and width  
pelvic part: + 11 cm  
Width strap: + 48 mm  
Buckle: 50 mm

Its shoulder heights are separately adjustable to help position and compensate for a bad position.

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
- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm (the backrest and seat of the chair must have a separation space).
- Waist (in cm)



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



CSYP2HP-PF

v 07.02/25

### Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



## SLIM FIT

### HOSPITAL RANGE

Combination of polyurethan foam  
and on both faces  
Jersey/PU coating  
100% polyester bias  
100% polyester strap  
Nylon buckle  
Thermo compressed foam

- Position the pelvic part on the seat of the chair and let the buckle go out towards the back of the chair.
- Let the patient sit on the chair.
- Position the belt around the patient's waist.
- Close the belt using the velcro strap.
- Place the chest belt being careful that the shoulders pads are positioned on the patient's shoulders.
- Pass the strap of the back part of the belt behind the backrest of the chair.
- Before closing it and without adjusting it, check that the back strap has a female buckle looking downwards and 2 female buckles looking upwards.
- Close the 4 rear buckles, adjust