

[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

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
This device is intended for lifts allowing to transfer a person in standing position.

Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fasteners

## PREMIUM RISER STRAP



- CE certified and manufactured in France, the Standard Sling adapts to the morphology of each patient by its multitude of sizes (S, M, L, XL) and is used with all stand-up lifts equipped with a hook system.
- It is placed under the armpits based on the device used, due to its full padding under the arms which distributes traction on the patient and avoids painful pressure.



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



SWYP3-PF

v 06.02/23

### Important Recommendations :


- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



## PREMIUM RISER STRAP

### SIZE & DIMENSIONS (in cm)

	Plastron	A	B
S	70 / 90	21	78
M	90 / 115	25	92
L	115 / 140	29	106
XL	140 / 165	33	120

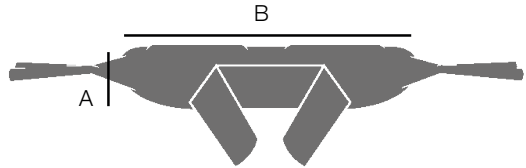

Composition  
100 % polyester fabric  
40 mm polyester strap - 1450 DaN  
50 kg/m<sup>3</sup> PE foam  
20/25 kg/m<sup>3</sup> PE foam  
20 mm polyester bias

Position the bottom of the backrest around the shoulder blades in order that the ears of the attachment system are on either side of the person.  
Place the fasteners of the backrest under the arms of the person, so that the beads are under the armpits.  
Close the plastron.  
Hook the two fasteners of the backrest to the stand-up lift hooks.

### SETTINGS

The fastening system, with multiple hangers, allows to adjust the position of the person.

### CAUTION

Read the instruction manual before using the slings.  
The backrest hangers must be hung on the same color.  
Never change the attachment setting of the ears to the stand-up lift hooks, when someone is installed in the sling.  
Before lifting a person with the stand-up lift, make sure the sling is securely hooked to the hooks.