



www.nausicaa-medical.com

This device is intended for lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

ENVELOPING SLING



- CE certified and manufactured in France, the Enveloping Sling is recommended for bed to chair to bed transfers and for "lifting from the floor" transfers in a lying position.
- The integrated and boned headrest in addition to the enveloping shape of the sling make it suitable for the transfers of agitated and anxious patients.
- Its U-shape avoids the placement of the sling under the patient's buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.



Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.

Remove the aluminium bar before putting the sling in the washing machine.



SLK-PF



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.



v 06.09/23

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.




ENVELOPING SLING

SIZES & DIMENSIONS (in cm)

	A	B	C
S	59	88	108
M	66	92	120
L	71	96	134
XL	77	101	150

Composition
100% polyester fabric
40mm polyester strap - 1450 DaN
50kg/m³ Polyester foam
Aluminium stiffener 15x3mm

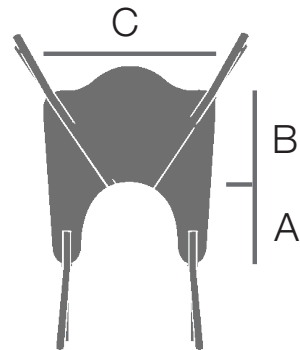

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar :

- 2 on both sides of the back
- 1 at each end of the thigh support.

Position the lower part of the sling at the coccyx so that the loops of the hanging straps are on both sides of the patient. Slightly raise the right leg of the patient, pass the right thigh support under the right thigh. Do the same for the left leg. Both hanging straps are positioned between the legs of the patient. Cross these hanging straps. Hang both hanging straps of the back and hanging straps of the thigh supports to the spreader bar.

ADJUSTMENTS :

The different loops on the hanging straps allow various sitting positions.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.