

www.nausicaa-medical.com

This medical device is intended for patient positioning. This is not a restraint system.

Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

## HEADREST



The removable TAMOV-PF headrest is used on the straps.

TAMOV-PF



The TABAL-PF removable and woven headrest is used on the straps.

TABAL-PF



The TAAT-PF headrest is used on the straps

TAAT-PF



Remove the aluminium bar before putting the headrest in the washing machine.



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.



TAMOV-PF / TABAL-PF / TAMAX-PF

v 06.09/23