

ISO 13485

BUREAU VERITAS
Certification



NAUSICAA

MEDICAL

www.nausicaa-medical.com

Made in France
since 1992



This medical device is intended for patient holding and positioning. It is not a restraint system.

Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

EVOLUTIVE ECO CHEST BELT



CECO3-PF-B

TO BE USED ONLY FOR ITS INTENDED
PURPOSE:
TO PREVENT FALLS



CECO3-MP

A bust belt plus a one-size-fits-all pelvic module allow you to switch from abdominal to full-body support as your needs evolve, with no size constraints.

- Maximum useful area: 228 cm



Use of the device requires the presence and vigilance of a third party. In the event of use by an agitated or disoriented person, supervision becomes imperative.



This medical device must only be handled by personnel trained (by a healthcare professional) and qualified for its use (intended purpose, installation and operating environment, etc.). If the medical device is incorrectly installed, there is a risk of injury to the patient.

CECO3-PF-B / CECO3-MP



v 06.02/25

Matériaux textiles
conformes à la législation
REACH
sur les substances indésirables

NAUSICAA Médical
12 Allée du Plot - ZA Pôle Actif
30660 Gallargues le Montueux
FRANCE



Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- fit the product as snugly as possible on the patient
- These products must not be placed in direct contact with damaged skin.



ECO EVOLUTIVE CHEST BELT

Composition
Jersey cotton polyester lined with polyurethane foam
Jersey 100% polyester
Bias 100% cotton
Straps 100% polyester
Nylon buckles

INSTRUCTIONS FOR USE

Do not use this belt if it does not fit the patient.

The eco evolutive chest belt is available in 2 modular variants

- 1 -The ECO Chest Belt (CECO3-PF-B) consists of the ECO Abdominal Belt and a chest Support
 - a - Position the belt at waist level.
 - b - Pass the straps behind the back of the chair.
 - c - Close the nylon buckle, then adjust the length of the straps, taking care not to compress the patient's abdomen.
 - d - Fit the shoulder harness, close the 50 mm buckle, position the shoulder straps on the patient's shoulders, with the buckles behind the back of the chair. Pass a shoulder strap under the abdominal strap, connect the buckles and adjust its length.

2 -The Integral ECO Belt (CECO-MP) consists of the Abdominal ECO Belt plus a Bust Support Module and a Pelvic Support Module

- a - To fit the Integral ECO belt, proceed first as for the Pelvic ECO belt (2), then as described in paragraphs d and e for the Bust ECO belt (3).



Model CECO3 + Bust Module



CECO3 Model + Bust Module + Pelvic Module