

ISO 13485

BUREAU VERITAS
Certification



NAUSICAA

MEDICAL

www.nausicaa-medical.com

Made in France since
1992



This medical device is intended for patient support & positioning. It is not a restraint system.

Before each use, it is important to check: the presence of the label; the stitching on the straps: that they do not fray or come undone; that the device shows no tears, snags, holes or signs of fraying (fabric and fastening straps); that the buckles are not cracked or show any weakness in their fastening.

ECO EVOLUTIVE INTEGRAL SUPPORT BELT



CECO3-PF-B

TO BE USED ONLY FOR ITS INTENDED
PURPOSE:
TO PREVENT FALLS



CECO3-PF-B + CECO-MP

A bust belt plus a one-size-fits-all pelvic module allow you to switch from abdominal support to full support as your needs evolve, with no size constraints.

- Maximum usable perimeter : 228 cm



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



CECO3-PF-B



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



v 06.02/25

Matériaux textiles
conformes à la législation
REACH
sur les substances indésirables

NAUSICAA Médical
12 Allée du Plot - ZA Pôle Actif
30660 Gallargues le Montueux
FRANCE



Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- fit the product as snugly as possible on the patient
- These products must not be placed in direct contact with damaged skin.



ECO EVOLUTIVE INTEGRAL SUPPORT BELT

Composition
Jersey cotton polyester lined with polyurethane foam
Jersey 100% polyester
Bias 100% cotton
Straps 100% polyester
Nylon buckles

INSTRUCTIONS FOR USE

Do not use this belt if it does not fit the patient.

The eco evolutive bust belt is available in 2 modular versions:

1 -The ECO Bust Belt (CECO3-PF-B) consists of the ECO Abdominal Belt plus a Bust Support Module.

- a - Position the belt at the patient's waist.
- b - Pass the straps behind the back of the chair.
- c - Close the nylon buckle, then adjust the length of the straps, taking care not to compress the patient's abdomen.
- d - Position the shoulder harness, close the 50 mm buckle, position the shoulder straps on the patient's shoulders, with the buckles behind the back of the chair.
Pass a shoulder strap under the abdominal strap, connect the buckles and adjust its length.

2 -The Integral ECO Belt (CECO-MP) consists of the Abdominal ECO Belt plus a Bust Support Module and a Pelvic Support Module.

a - To fit the Integral ECO belt, proceed first as for the Pelvic ECO belt (2) and then as described in paragraphs d and e for the Bust ECO belt (3).



Model CECO3 + Bust Module



CECO3 Model + Bust Module + Pelvic Module