



www.nausicaa-medical.com

This device is intended for lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

STANDARD HAMMOCK SLINGS



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
Compatible with
removable headrest
(TAMOV).



- CE certified and manufactured in France, the Standard Comfort Hammock is recommended for bed to chair transfers where the sling remains on the chair.
- Its simple and indented cut on the sides makes it the preferred sling to remain under a patient in a chair, as it shows no thickness and folds that cause decubitus ulcers and discomfort.
- The leggings comfort foam reduces the shearing effect and the installation must be done in bed in a lying position at 180° by practicing a lateralization.
- Moreover, this sling avoids hips tightness.



Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.



SHA2-PF / SHAC2-PF / SHA2BA-PF / SHAC2BA-PF

v 06.02/23

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




STANDARD HAMMOCK SLINGS

SIZES & DIMENSIONS (in cm)

	A	B	C
S	57	62	54
M	61	67	65
L	65	69	76
XL	70	71	86

Composition
100% polyester fabric
40mm polyester strap 1450 DaN

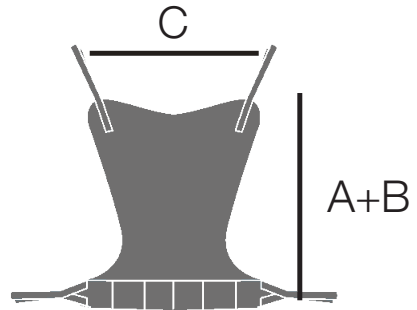

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

This sling has a 4 points hanging system to the spreader bar :

- 2 on the top of the sling (at the shoulders' level)
- 2 on the bottom of the sling (at thigh level)

Position the lower part of the back at the coccyx so that the loops of the hanging straps are on both sides of the patient.

If need be, position the lower part of the head pad under the patient's shoulders (the top

of the head pad positioned on the top of the head). The head pad will be maintained by the shoulders pressure.

Hang each hanging straps on the spreader bar without crossing.

ADJUSTMENTS :

The multiple hanging straps allow various positions : sitting, semi-sitting or lying position.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.