www.nausicaa-medical.com

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www.nausicaa-medical.com

Before each use, it is important to check: the presence of the label, that the device does not show any tears, snags, holes or signs of fraying and the good condition of the hook and loop.

HEEL PADS

HEEL PADS SMALL MODEL

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
HELP TO PREVENT
PRESSURE SORES
ON HEELS

HEEL PADS LARGE MODEL

HELP TO PREVENT
PRESSURE SORES
ON HEELS

Description :

The Heel Pads allow to to relieve pressure points and to wrap the foot in a furtrim, preventing rubbings on the heels and malleoli.

The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.

LM2TAL-PF

v 06.02/23

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.

HELL PADS

COMBINATION OF POLYURETHAN FOAM
AND JERSEY/COTTON POLYESTER FABRIC
JERSEY 100% POLYESTER FABRIC
100% POLYESTER FUR
100% POLYESTER VELCRO STRAP

- Put the heel, fur side, right in the bottom of the heelpad.
- Close the heelpad on the foot through the velcro strap(s).