

www.nausicaa-medical.com

This device is designed for stand up lifts used to transfer a person in a standing position.

Before each use, it is important to check: that the label is attached, that the straps are not frayed or coming undone, that there are no tears, snags, holes or signs of fraying (fabric and attachment straps), and that the buckles are not cracked or have weak attachments.

DORSAL SLING XXL



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
The sling is positioned at waist level, and is adjusted by tensioning the sling with the patient's back against the backrest.

the patient's back against the backrest.

The hands are placed on the handles and the arms stretched out to tone the upper body.

When the lift arm is fully extended, the patient is in a standing position. Partial deployment of the lift arm combined with the buttock sling gives a seated position.

Its straps help to secure the patient when walking.



Use of the device requires the presence and vigilance of a third party. If used by a person who is agitated or disorientated, supervision becomes imperative.

This medical device must be operated or trained by a healthcare professional.



SWYP3-PF

v 02.12/24

Important recommendations

- To ensure maximum effectiveness when using the products in this range, it is essential to:
 - choose the right size for the patient
 - fit the product as snugly as possible on the patient
- These products must not be placed in direct contact with damaged skin.




DORSAL SLING XXL

SIZES & DIMENSIONS (in cm)

2XL	A	145	4XL	A	196
	B	33		B	33
3XL	A	172			
	B	33			

Composition

Jersey 100 % polyester
Sling polyester 40 mm - 1450 DaN Sling polyester 50 mm - 2500 DaN Foam PE 50 kg/m³
Foam PE 100 kg/m³
Foam polyurethane 20 kg/m³
Foam polyurethane 18 kg/m³
Coating polyurethane on polyester
Bias polyester 40 mm
Bias polyester 20 mm
Self grip 100 % polyester

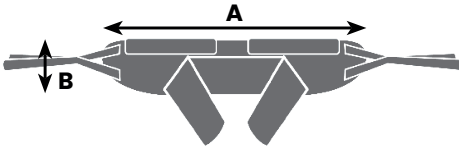

Position the bottom of the backrest at lumbar level so that the ears of the attachment system are on either side of the person. Pass the backrest fasteners under the person's arms, so that the backrest is in place. Close the breastplate.
Hook the two backrest attachments to the hooks on the sit-to-stand lift (mobile lift).

ADJUSTMENTS

The multiple-hanger system allows the patient's position to be adjusted.

PRECAUTIONS FOR USE

Read the instructions for use before using the straps.

The backrest lines must be attached to the same colour.

Never adjust the hooks on the mobile lift when a person is sitting in the sling.

Before straightening a person with the mobile lift, make sure that the strap is properly hooked.

INSTALLATION OF DOUBLE STRAP KITS FOR AMBULATION

The double gluteal strap kit can also be installed in ambulation mode. To do this, the straps must be installed as follows:

Clip the 50 mm male-female buckles on one side of the strap, pass the strap under one leg and then clip the strap on the same side. Repeat on the other side.