



www.nausicaa-medical.com

This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

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
HAMMOCK WITHOUT CROSSING SLINGS



- CE certified and manufactured in France, the Comfort Hammock Without Crossing is recommended for bed to chair to bed transfers for amputees and for "lifting from the floor" transfers in a lying position with the addition of headrest.
- It is a hammock sling with a U-shape which avoids the placement of the sling under the patient's buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.
- Moreover, this sling avoids hips tightness and the foam leggings reduce the pressures exercised on the thighs.



Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



SUCPEHA2-PF / SUCBAHA2-PF

v 06.11/23

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.




HAMMOCK WITHOUT CROSSING SLINGS

SIZES & DIMENSIONS (in cm)

	A	B	C
S	41	53	75
M	46	57	83
L	50	61	92
XL	55	66	102

Composition
Jersey 100 % polyester
Sangle polyester 40 mm - 1450 DaN
Mousse PE 50 kg/m³
Biais polyester 40 mm

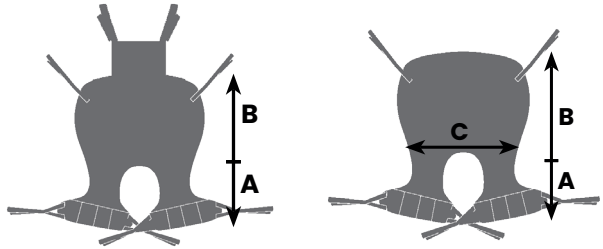

Place the patient in recovery position. Fold the hammock in 2 part by respecting the vertical axis.

- Position the fold according the vertical axis in the middle of the patient's back.
- Put the half sling on the patient's free side.
- Fold the extreme side of the sling supporting by the patient
- Reverse the patient on his back with the half sling places on the free side
- Recover the patient on the sling on side previously available
- Release the part previously folded and spread it
- Place the patient on his back on the sling deployed
- Hook the top lines and the bottom lines in the middle of the patient hoist

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.