# NAUSICAA

---

## MEDICAL

[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

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# Content

## PCE-CS 300 weighing system

• Regulatory Data	18
• Safety Instructions	19
• Inspection, servicing and maintenance	22
• Technical features	23
• Packing Content	23
• Icons Descriptions	24
• Buttons Descriptions	24
• Settings and Use	25
• Weighing Procedures	27
• Warranty	29

# Regulatory data

---

This measuring instrument is not a medical device or an accessory for medical devices. The information displayed is for guidance only. This weighing system must be used for weighing purposes only (as a bathroom scale) and must be removed before transfer.

It complies with the following standards:

- CE marking
- Directive 2014/30/EU (EMC)
- EN 55014-1 +A1 +A2 and EN 55014-2
- EN 61000-3-2 and EN 61000-3-3 and EN 61000-6-2 and EN 61000-6-4

Static strength and stability tests have been carried out in accordance with ISO 10535 to ensure safe use and compatibility with Nausicaa medical lifts: Nausifyly 3 and Flyer XL.

# Safety Instructions

**Intended use:** To assess the weight of people with reduced mobility.

Always comply with the regulations in force when using electrical components in accordance-safety requirements.

- Incorrect installation will invalidate the warranty.
- This scale has been designed for indoor use.
- Always observe the permissible ambient operating temperatures.
- Do not exceed the maximum values specified in the applicable standards.
- Do not overload the scale. Overloading will be indicated by an audible signal.
- In the event of a problem, contact Nausicaa Médical after-sales service:

**Telephone : 04 66 71 71 80**

**Fax : 04 66 71 71 81**

**Mail : sav@nausicaa-medical.com**

Please read this user manual carefully and in its entirety before using the appliance for the first time. This appliance must only be used by qualified personnel. No liability will be accepted for damage caused by failure to observe the warnings in the operating instructions.

- If you are not going to use the device for some time, remove the batteries.
- **This device must only be used as described in this user manual. Failure to do so may result in dangerous situations.**
- Do not use this device unless the ambient conditions (temperature, humidity, etc.) are within the limits indicated in the specifications. Do not expose the device to extreme temperatures, direct sunlight, extreme humidity or place it in wet areas.
- Do not expose the device to strong shocks or vibrations.
- Only qualified personnel from PCE Instruments may open the casing of this device.
- Never use this device with damp or wet hands.
- Do not make any technical modifications to the device.
- **This device should only be cleaned with a damp cloth. Do not use abrasive or solvent-based cleaning products.**
- **The device must only be used with accessories or spare parts supplied by NAUSICAA MEDICAL.**
- **Before each use, check that there is no visible damage to the casing. damage. If there is, do not use the device.**
- The measurement range indicated in the specifications must never be exceeded.
- Failure to comply with the safety instructions may result in injury to the user or patient and damage to the device.
- Since lifting loads is associated with risks, both the manufacturer and the user must comply with certain obligations in order to avoid them. The load cell must never be used in a hazardous manner.
- **The person responsible for the equipment, as well as the user, must observe and comply with national standards, such as accident prevention standards, safety standards, working standards and all other standards applicable to the use of the load cell.**
- When using the equipment, other directives must also be taken into account, such as the standards and safety information provided by the manufacturer of the load cell, the manufacturer of the patient lift, the manufacturer of the lifting strap, etc.

# Safety Instructions

---

- **In this context, only trained personnel may install, start up, maintain and operate the appliance.**
- The appliance must not be modified and must only be used for its intended purpose.
- **Regularly inspect the appliance and all other components (e.g. pins, clasps, clamps, flail, strap, etc.) and record the relevant notes in the maintenance booklet. Before each use of these components, carry out at least one thorough visual inspection. If any of these components are incomplete or damaged (e.g. cracks, deformations, cracks, missing clasp, etc.), they must not be used. In this case, it is essential to consult your establishment's safety officer.**
- Repairs may only be carried out by trained personnel, and only approved spare parts may be used. All repairs, maintenance and spare parts must be documented by the service engineer.
- The user manual and safety instructions must always accompany the appliance. A QR code is placed on the scale by means of a label that refers to this user manual.
- **The scale may only be used for weighing patients. Torsion of the load (torsional stress) is not permitted. It is therefore forbidden to transfer a person, to exert a transverse force, to move the lift when the patient is suspended, etc.**
- **Do not exceed the load capacity [MAX] of the load cell or lift used.**

# Safety Instructions

## IMPORTANT WARNINGS

	<p>The PCE-CS 300 scale must be installed by qualified personnel and it must be ensured, before use, that the various components - lift, scale and straps - are correctly assembled so that the patient can be weighed in complete safety; otherwise there is a risk of falling and serious consequences for the patient.</p>	
  	<ul style="list-style-type: none"> <li>* <b>AFTER THE ASSEMBLY PHASE AND BEFORE WEIGHING THE PATIENT, THE OPERATOR MUST ENSURE THAT: THE PATIENT IS PROTECTED BY SOFT ELEMENTS TO AVOID ANY SHOCK OR SERIOUS DAMAGE. IN PARTICULAR, PLACE MATTRESSES, CUSHIONS, A BED OR ANY SOFT ELEMENT UNDER THE PATIENT TO ENSURE MAXIMUM SAFETY DURING SUSPENSION WEIGHING.</b></li> <li>* The PCE-CS 300 load cell and the straps must be in a vertical position during use.</li> <li>* During lifting and weighing of the patient, the operator must ENSURE that the lift (with the associated scale) is stationary on a flat surface.</li> <li>* <b>It is strictly forbidden</b> to perform a transfer when the load cell is mounted on the lift. It is also forbidden to exert a transverse force or to move the lift when the patient is suspended.</li> <li>* The operator <b>ALWAYS</b> accompany and hold the patient's sling when lifting the patient, in order to avoid twisting and sudden movements of the load cell that could cause breakage and irreversible damage, with the risk of serious consequences for the patient.</li> <li>* <b>DO NOT MOVE, CARRY OR TURN</b> the lift with the scale when the patient is suspended.</li> <li>* <u>When the weighing phase is over, remove the patient from the sling and move them safely with other devices</u></li> </ul>	
 <p>INTERDICTION DE TORSION</p>	<p>During the weighing phase with the PCE-CS 300, before releasing the patient strap to measure the exact weight, the operator must ensure that the patient remains motionless to avoid twisting or oscillation of the device and incorrect weighing.</p>	
 <p>INTERDICTION DE TORSION</p>	<p>After the weighing phase, the operator must ALWAYS accompany and hold the patient's sling in order to avoid twisting or sudden movements of the scale and sling that could cause breakage and irreversible damage, with serious consequences for the patient.</p>	
<p><b>CAUTION</b> To weigh the patient <b>ALWAYS UNDER SAFE</b> the operator <b>MUST</b> place a cushion under the lifted patient.</p>		<p> INTERDICTION ABSOLUE  DE DÉPLACER ET TRANSPORTER <b>LE PATIENT EN SUSPENSION</b></p> 

# Inspection, servicing and maintenance

## A. Daily check of the scale before use

Before each use of the scale, it is recommended that you check :

- There is no visible deformation or damage (e.g. cracks, etc.).
- That all installed components are complete and in perfect condition (e.g. pins, clasps, beam, straps, etc.).
- That the entire system does not represent any danger (e.g. incorrect assembly, etc.)

## B. Cleaning

The scale should be cleaned using a soft cloth dampened with water or a neutral detergent, avoiding the use of solvents or abrasive substances.

Do not use large quantities of water when cleaning the instrument, as this could damage the electrical components.

### **Always disconnect the instruments from the power supply before carrying out cleaning interventions**

If the instrument is not to be used for a long time, remove the batteries from the terminal and cover the instrument to protect it.

During transport, take care not to subject the instrument to excessive shock or mechanical stress.

For repairs or assistance, please contact your retailer or Nausicaa Médical after-sales service.

## C. Regular inspection and maintenance :

Every 3 months:

- Check the degree of wear of accessories.
- Check all dimensions.
- Check for any deformation or damage.
- Check that all components are working properly.
- Check the gap between the flanges and the load cell housing (approx. 1 mm). If more, danger!

Every year:

- Check all components that support the patient, e.g. check for hairline cracks, etc. This maintenance must be carried out by qualified personnel.
- Check the operation of the load cell: Check the mechanical components, check the operation of the buttons, check the stability of the load cell.

Every 3 years:

- Change all the pins and flanges

Every 5 years:

- Replacement of all charging elements.


# Technical Specifications


<b>Weighing range</b>	300 kg
<b>Resolution</b>	0.05 kg
<b>Minimum weight</b>	2 kg
<b>Reproducibility</b>	0,1 kg
<b>Max. overload</b>	150%
<b>Stabilisation time</b>	< 10 seconds
<b>Tare</b>	In the range
<b>Protection</b>	IP54
<b>Screen</b>	LCD with backlighting
<b>Digit height</b>	20 mm
<b>Power supply Scale</b>	4 x batteries 1.5V type AA
<b>Weighing range</b>	0 ... +40 °C / <90 % H.r.
<b>Resolution</b>	1.2 kg (batteries included)

## Packing Include

- 1 weighing scale PCE-CS 300
- 1 pin with groove
- 1 pin clasp
- 4 x 1.5V AA batteries
- 1 user manual, which can be consulted using the QR Code label affixed to the weighing system.
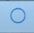
# Icons Descriptions

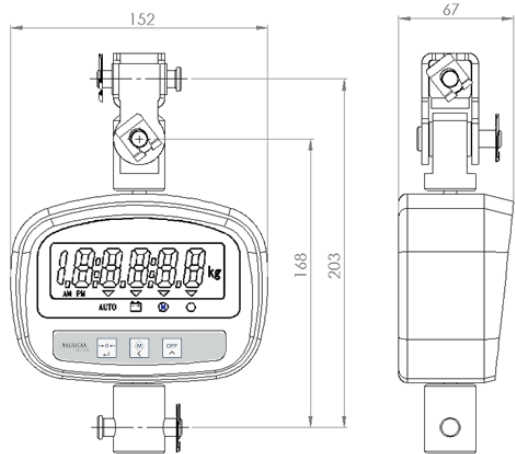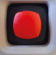
	Weight is stable
	If the arrow above this symbol appears, the reading capacity (resolution) increases to 0.1 kg.
	If the arrow above this symbol appears, you need to change the batteries.
	If the arrow above this symbol appears, the display backlight is activated during the weighing process.

ENGLISH

# Buttons Descriptions

## Weighing scale buttons

	Tare
	Setting accuracy : 0.1 kg or 0.05 kg
	Feature 1: Allows you to put the device to sleep by pressing twice. Note that this standby mode continues to consume energy, so it's best to switch the device off using the red button on the back of the scale. Feature no. 2: Allows you to change the number when setting the sub-menu.
	Switching the scale on and off (located at the rear)

# Settings and Use


## Setting up

Remove the flail from the lift arm.

**1. Fit the adapter kit:** Position the 2 bearings on the blue ball-and-socket joint, present it inside the arm. Position the fabric protection and place a pin through it. Lock the pin in place with the clasp.

**2. Mount the load cell on the ball joint:** Position the load cell under the ball joint, fit the 2 plastic rings on either side of the flange and place the grooved pin. Secure the pin with the clasp.

**3. Fit the beam to the load cell:** Position the beam shaft under the load cell. Check that the anti-friction washer is in place. Place the pin in the groove. Lock the pin in place with the iron.

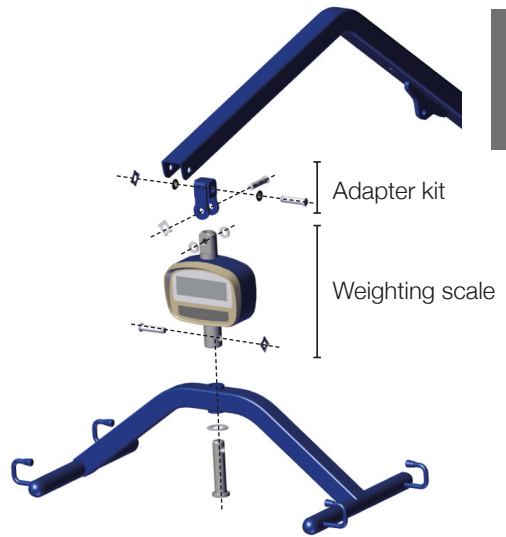ENGLISH

**WEIGHING SCALE READY**

# Settings and Use

---

## Getting started

1. Remove the battery compartment cover.
2. Insert 4 x 1.5 V, type AA batteries (supplied) and turn the switch to position 'B'. If you have chosen the balance with optional lithium battery, turn the switch to position 'A'.
3. Close the battery compartment.
4. Press the mechanical on/off switch on the back of the scale. The maximum load will appear on the display and the scale will perform a self-test.
5. The display then shows 0.00. When the corresponding symbol is displayed, the scale is ready for use. We recommend a warm-up time of 3 to 5 minutes before starting to weigh.

## Zeroing

The scale usually displays 0.0 kg or 0.00 kg when it is switched on. If the display shows a different value, press the 'Zero' button to zero the display.

## Backlight (see p.26)

1. If, in the sub-menu, the 'Auto' function is set to 0, the backlight is activated as soon as you press the 'Zero' button for more than 3 seconds. Hold down the 'Zero' button for about 3 seconds to deactivate the backlight.  
If the 'Auto' function is set to 1 in the sub-menu, the backlight is automatically activated when the scale is charged and deactivated when it is uncharged.

# Weighing Procedures

Before using the scale, make sure that the display shows '0.00'. If it does not, press the 'ZERO' button. The scale has been designed to take measurements when the weight is stable.

**WARNING**

INTERDICTION DE TORSION

- An inclination greater than 3° will result in inaccurate readings.
- For correct weight measurement, during the weighing phase, before releasing the patient's strap, the operator must ensure that the patient remains still in order to avoid twisting or oscillation of the device and incorrect weighing.
- It is advisable not to apply the load cell to lifts with a fixed suspension arm that does not allow 360° rotation.
- Do not tilt or rotate the load cell horizontally.

<p><b>WARNING</b></p> <p><b>To weigh the patient ALWAYS IN SAFE CONDITIONS</b> The operator <b>MUST</b> place a cushion under the patient being lifted.</p> <p><b>To weigh the patient, scrupulously follow the instructions given in paragraph 3 Safety.</b></p>	 <p>COUSSIN DE SÉCURITÉ</p>	<p><b>⚠ INTERDICTION ABSOLUE ⚠</b> <b>DE DÉPLACER ET TRANSPORTER</b> <b>LE PATIENT EN SUSPENSION</b></p> 
---	--	---

1. To tare the scale, place the empty strap in suspension and press the 'Zero' button to set the display to zero. The weight displayed during weighing will therefore be the patient's net weight. This weight is only changed when the tare is changed or the scale is switched off.

2. Once the patient is suspended, the display will show the weight. When it indicates that the load is stable, you can read the patient's weight. If activated (see p.26), a beep will sound.

## Sub-menu settings

1. Switch on the scale.

2. While the scale is counting down from 99999 to 00000, confirm with the key 

3. The screen shows '00000'. Now enter a code to enter the sub-menu. Use the following keys:

 Change number

 Confirm the value entered and move on to the next position. The access code is '11111'.

Once entered, confirm with  .

# Weighing Procedures

4. The sub-menu is structured as follows:

Sub-menu	Setting Value	Effect
bEE	0	Activate beep when value stabilises
	1	Deactivate beep when value stabilises
Stb	0	Activate zeroing of unstable
	1	values
CHt	0	Deactivate zeroing of
	1	unstable values
Aut	0	Unit: lb
	1	Unit: kg
OFF	0	Manual backlight
	1	Automatic backlight
YES	0	Do not save
	1	Save

Use of buttons in the sub-menu :

Change the value.

Confirm the value entered and move on to the next position.

## Switching off the scale

1. Press the red 'on/off' button on the back of the scale to switch it off completely and prevent the battery from discharging.

2. When you have finished weighing, dismantle the scale to allow the patient to be transferred. The adaptation kit can remain permanently on the lift. The beam axle can be attached to it for transfers.

# Warranty

• Article 1 : NAUSICAA Médical S.A.S guarantees this appliance against all manufacturing and assembly defects in its mechanical and electrical components, and only for appliances used under the conditions stipulated by NAUSICAA Médical S.A.S.

The guarantee includes the electrical parts, unless broken.

This guarantee, the conditions of which are defined below, is valid for 24 months from the date of first departure from NAUSICAA Médical S.A.S.

• Article 2 : The warranty entitles the purchaser to free labour and the replacement, free of charge, of any parts found to be defective.

• Article 3 : The reseller is responsible for the carriage of the appliance and all related costs. Goods always travel at the risk and under the responsibility of the retailer.

Under guarantee: the cost of returning the goods after repair will be borne by NAUSICAA Médical S.A.S. Out of guarantee: the cost of returning the goods will be borne by the retailer, whether or not he accepts the repair estimate.

• Article 4 : The guarantee does not apply if the claims are the result of :

- An accident, misuse of the appliance or negligence on the part of the purchaser.

- Transport of the appliance without adequate protection.

- Modifications or alterations not approved by NAUSICAA Médical S.A.S.

- The impact of external factors (natural disasters, fire, impact, humidity, flooding, lightning, etc.).

- Installation and/or use that does not comply with technical and safety standards if the appliance is operated in a country other than the country of purchase.

- Failure to carry out routine maintenance.

- Use of chemical products

- Electrical or electrochemical interference

• The retailer may not invoke the benefit of the guarantee:

- If the serial number of the appliance has been removed, altered or rendered illegible.

- If the appliance under guarantee has been modified without the approval of NAUSICAA Médical S.A.S.

• Article 6 : No equipment will be loaned during the repair of defective equipment.

• Article 7 : All warranty claims must be made through the retailer. If this is not possible, the purchaser may send the equipment directly to NAUSICAA Médical S.A.S. In this case, the purchaser must indicate the retailer's contact details and a copy of the purchase invoice in a letter enclosed with the equipment.

• Article 8 : Spare parts under guarantee will only be sent after consultation with NAUSICAA Médical S.A.S. After Sales Service.

It should be noted that defective spare parts must be returned to the NAUSICAA Médical S.A.S. After-Sales Service Department, failing which they will be invoiced 1 month after the parts have been sent by NAUSICAA Médical S.A.S.

• Article 9 : Defective parts changed under or outside the guarantee will be guaranteed for 6 months from the date of repair or dispatch of spare parts.

• Article 10 : No retailer may unilaterally modify the terms of this warranty.

# NAUSICAA

MEDICAL

Export Service HeadOffice

Phone : 04 66 51 50 80

Fax : 04 66 51 50 47

Mail : [contact@nausicaa-medical.com](mailto:contact@nausicaa-medical.com)

[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

Your contact :

Distributed by NAUSICAA Médical S.A.S.

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